

Download and install software from the web.

Windows 7, 8 and 10 are restrictive about download and installation of software, even when the software is certificated.

This short manual will explain in detail how you can safely download and install our software or any other Windows software with a valid certificate.

MCT Ganseman BVBA
Steenweg 278
9470 Denderleeuw
Belgium
info@ganseman.com
www.ganseman.com

No copyright

Download and installation of software

We give a full explanation for the audiometry software. Even for users without an “Audiotest” apparatus, it is worth to download and install this software.

This guide was made with Internet Explorer. Other browsers behave similar.

Visit our site <http://www.ganseman.com>

Click in the left marge on “Software” and then “SetupAudioTest.zip”


The screenshot shows the MCT website interface. At the top, there are language options: [Nederlands](#), [English](#), and [Français](#). The main heading is "Windows 7, 8, 10 software for 32 en 64 bits". Below this, there are sections for "Ecg", "Monitor", and "AudioTest". In the "AudioTest" section, the link [SetupAudioTest.zip](#) is highlighted with a red box. To the left of the main content is a sidebar with several buttons: "Ecg apparatus L12", "Ecg apparatus F12", "Ecg Library", "Ecg Quiz", "News", "AudioTest", and "Software". The "Software" button is also highlighted with a red box.

Various possibilities exists for download and installation. We explain only one of them.


The screenshot shows a file download dialog box. The text reads: "What do you want to do with SetupAudioTest.zip (8.12 MB)? From: ganseman.com". There are three buttons: "Save", "Save as", and "Cancel". The "Save" button is highlighted with a red box.

Select “**Save**”


The download will take a few seconds, depending on the speed of the internet connection.

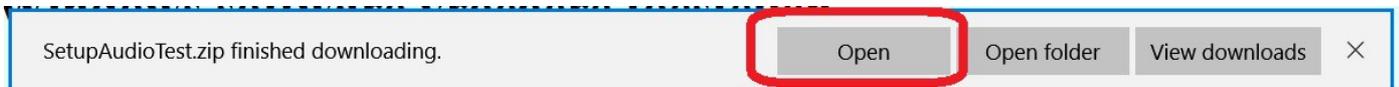

Select **“Open”**

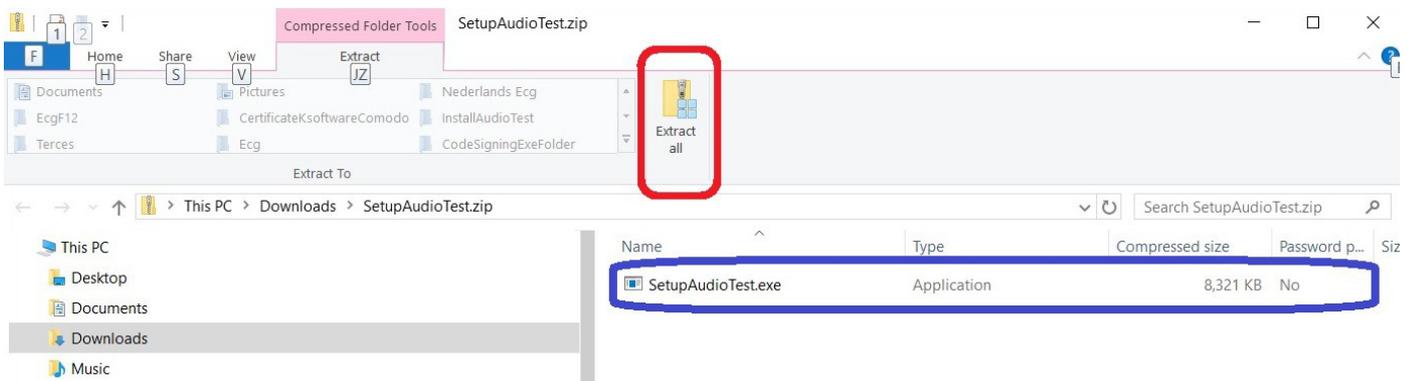


The download is a zipped, compressed file. In Windows Explorer select **“Extract all”**

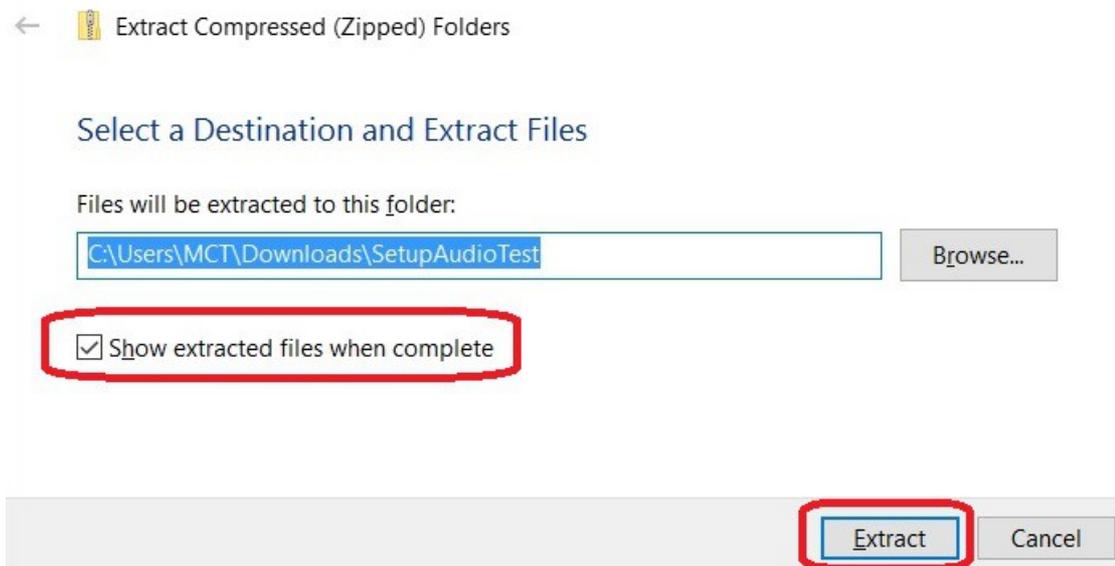

The name of the directory for extraction has no importance.


Check **“Show extracted files when complete”** and then button **“Extract”**

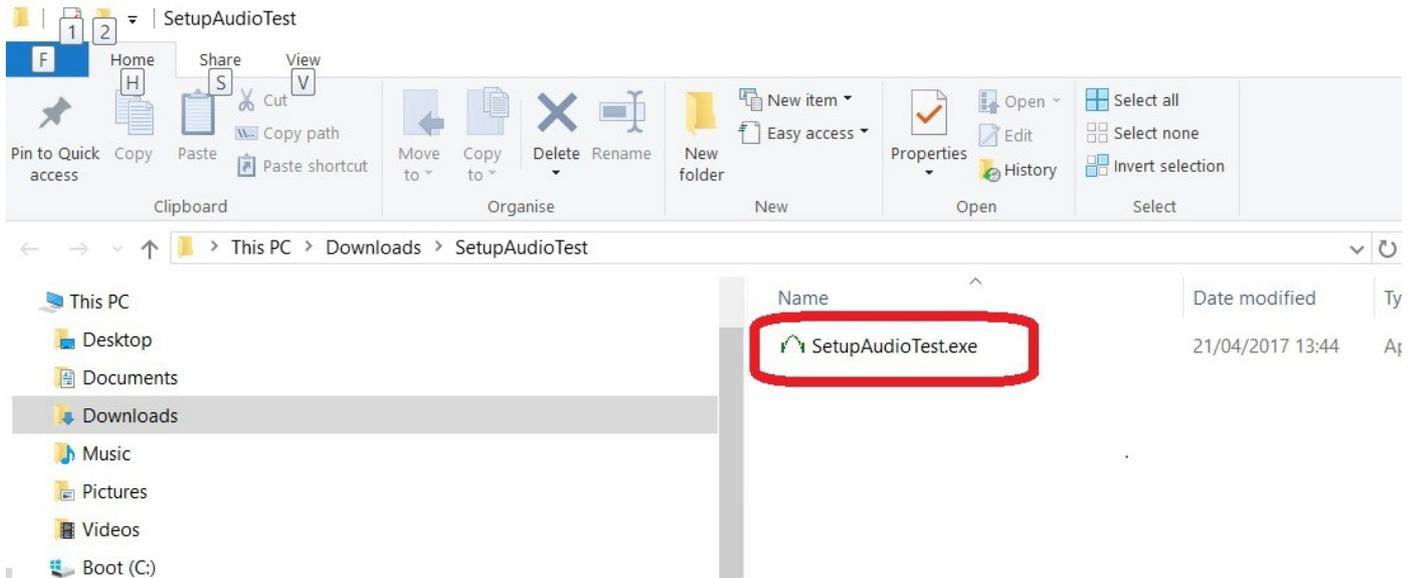


The extracted “SetupAudioTest.exe” becomes visible with his **icon** displayed at left.

Although it is possible, it is better not to start yet SetupAudioTest.exe.

Click with **right mouse button** on 'SetupAudioTest.exe' :

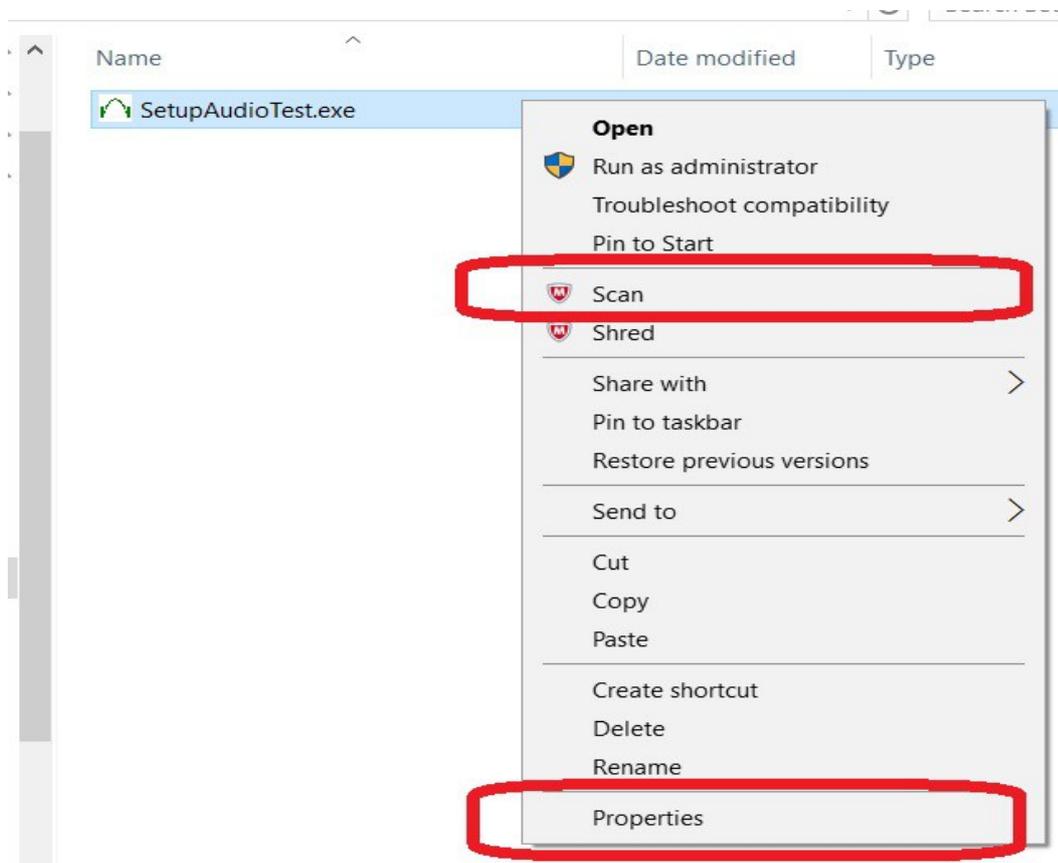

Clicking with **right** mousebutton on “SetupAudioTest.exe”(1) allows two important actions: scan for viruses with default virus scanner and control of the software certificate. After scanning click again “**Properties**” and then tab “**Digital signatures**” (2) The tab “Digital Signatures” is not visible when the software has no certificate.

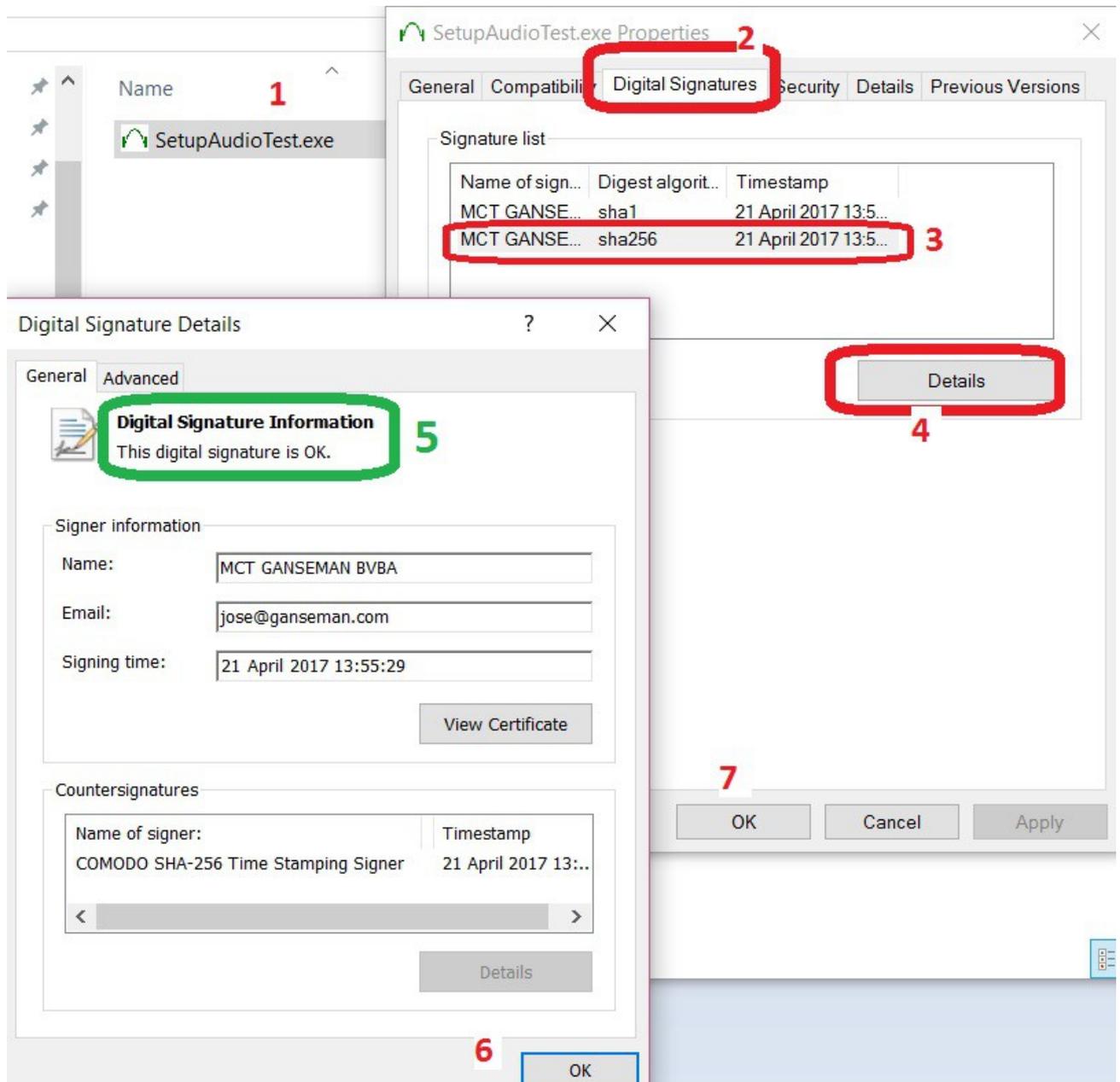

3 : 2 certificates are displayed. “sha1” is only important for operating systems lower than Windows 7. Select 'sha256' and then details (**4**). The result is displayed in (**5**) **6** and **7** will close the certificate screens.


A valid software certificate makes sure

- 1/ The software comes from the software publisher.
- 2/ The download is not changed after his upload.

Important – unblock.

After scanning for viruses and verification of the software certificate, click below

'Properties' Tab “General”

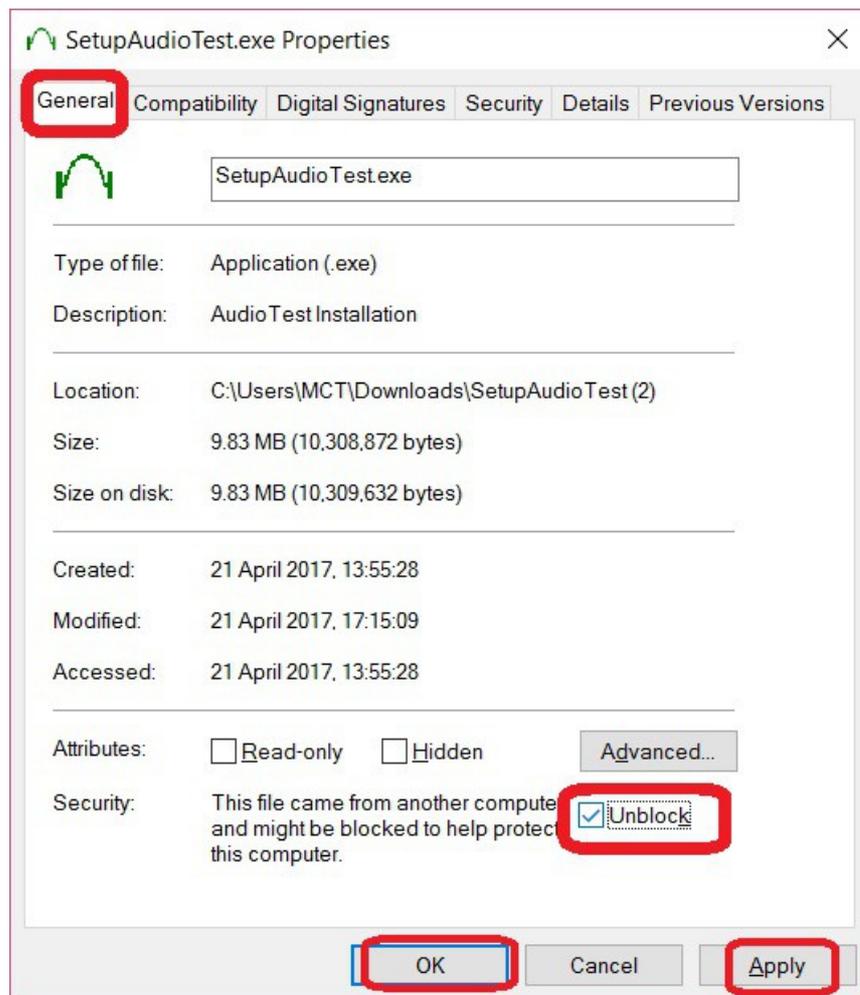

Check “Unblock” - Apply- Ok.

You may now start “SetupAudioTest.exe” in the Windows explorer. Double click it.

Intermezzo:

What happens if you don't check "Unblock" in Tab -General (see previous page) ?

After launching of 'SetupAudioTest.exe', most Windows versions, dependant on the setup, show the terrifying and horrible red 'SmartScreen' below :

| Name | Date modified | Type | Size |
|--------------------|------------------|-------------|-----------|
| SetupAudioTest.exe | 21/04/2017 13:44 | Application | 10,057 KB |

Windows protected your PC

Windows Defender SmartScreen prevented an unrecognized app from starting. Running this app might put your PC at risk.

[More info](#)

Don't run

Even then, there is a way out, click '**More info**'

Windows protected your PC

Windows Defender SmartScreen prevented an unrecognized app from starting. Running this app might put your PC at risk.

Application: SetupAudioTest.exe

Publisher: BE, 9470, OOST VLAANDEREN, DENDERLEEuw, STEENWEG 278, MCT GANSEMAN BVBA, N

Run anyway


Don't run

Verify the publisher and click 'Run anyway'


End intermezzo.

Finally you may start the installation of the software itself, mostly by clicking next.

Launch SetupEcg.exe, select language for the installation.

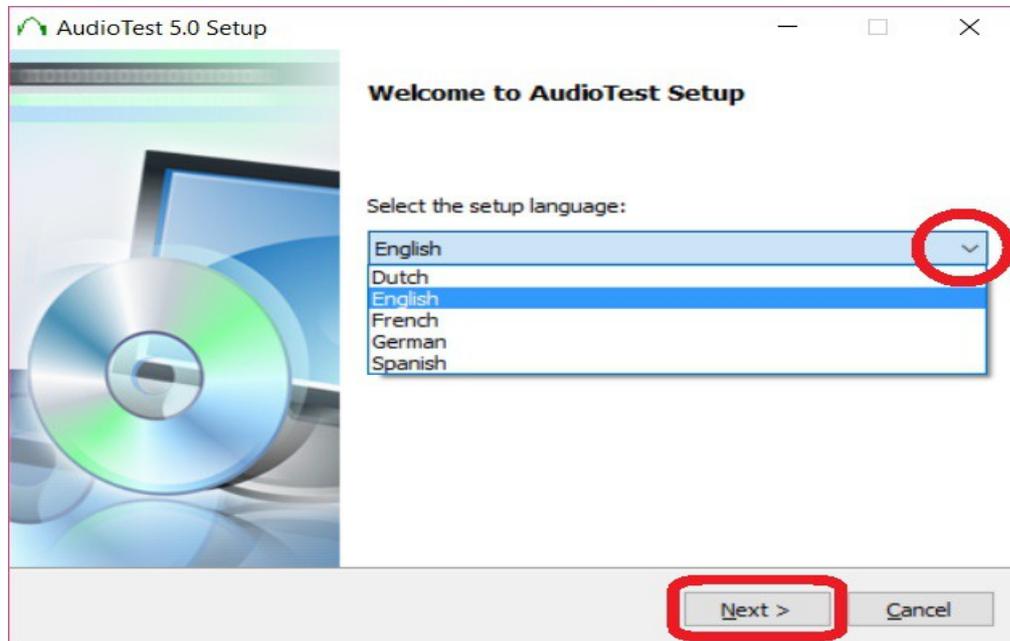


And then 'Next'

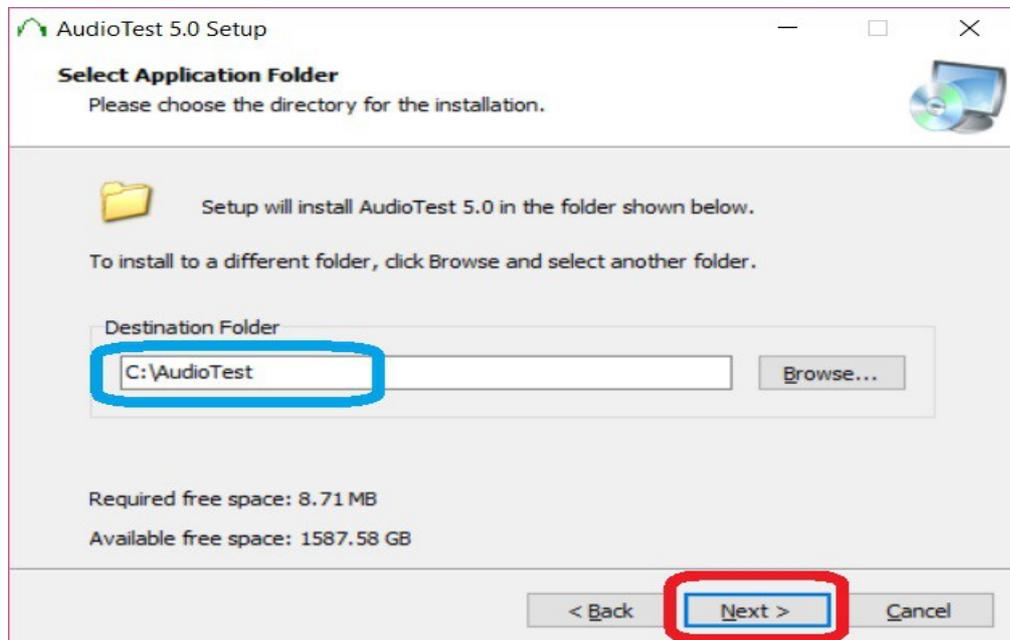

Default installation directory is C:\AudioTest. Installation in any other directory is possible like (C:\Program Files(x86)\AudioTest). When you install in another directory as C:\AudioTest enter the name of it in the setup of AudioTest.exe.

“Next”

Shortcuts in Start Menu and on the Desktop facilitate launching of the program.

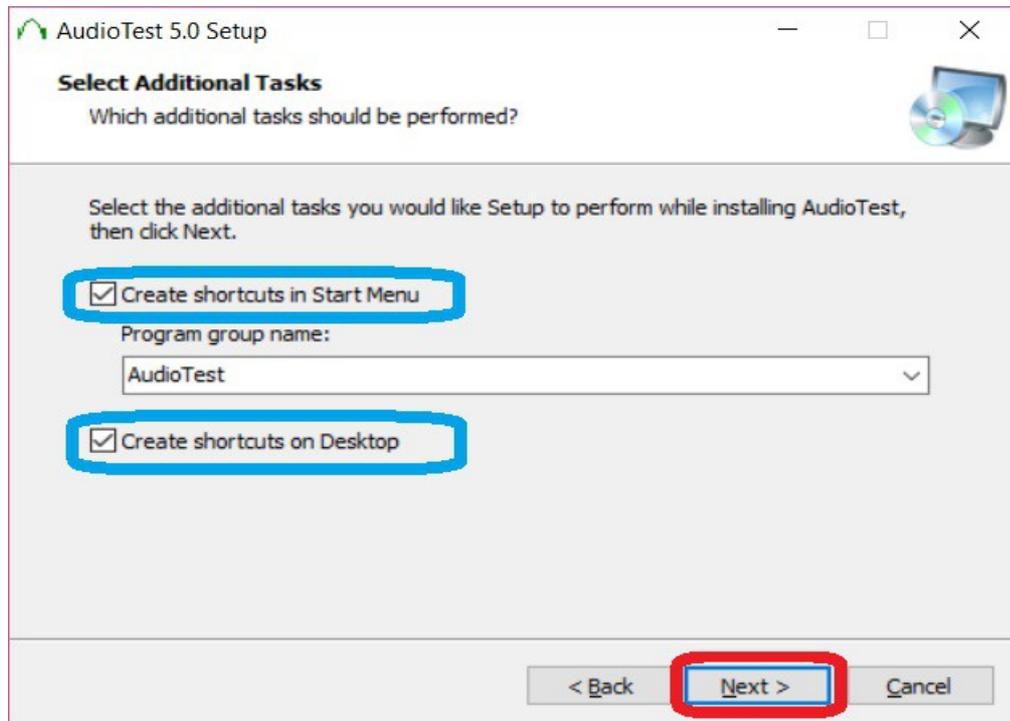


Installation of driver for USB serial converter is only necessary when you have an apparatus:




“Extract”



Check 'I accept this agreement'

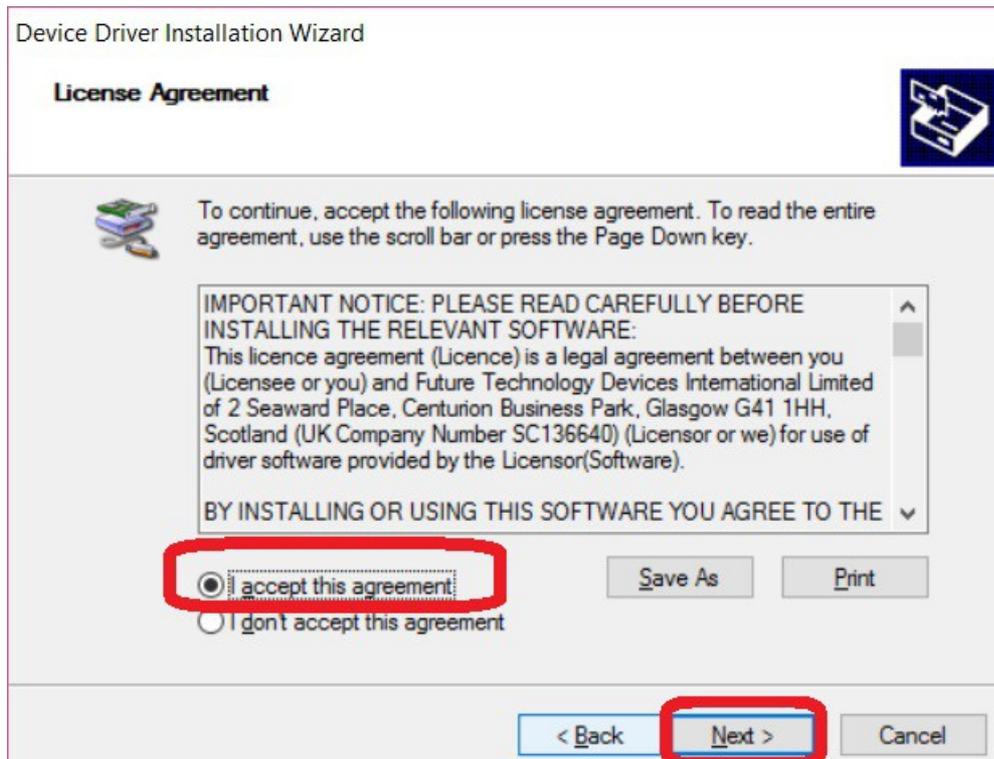

The installation is ready.

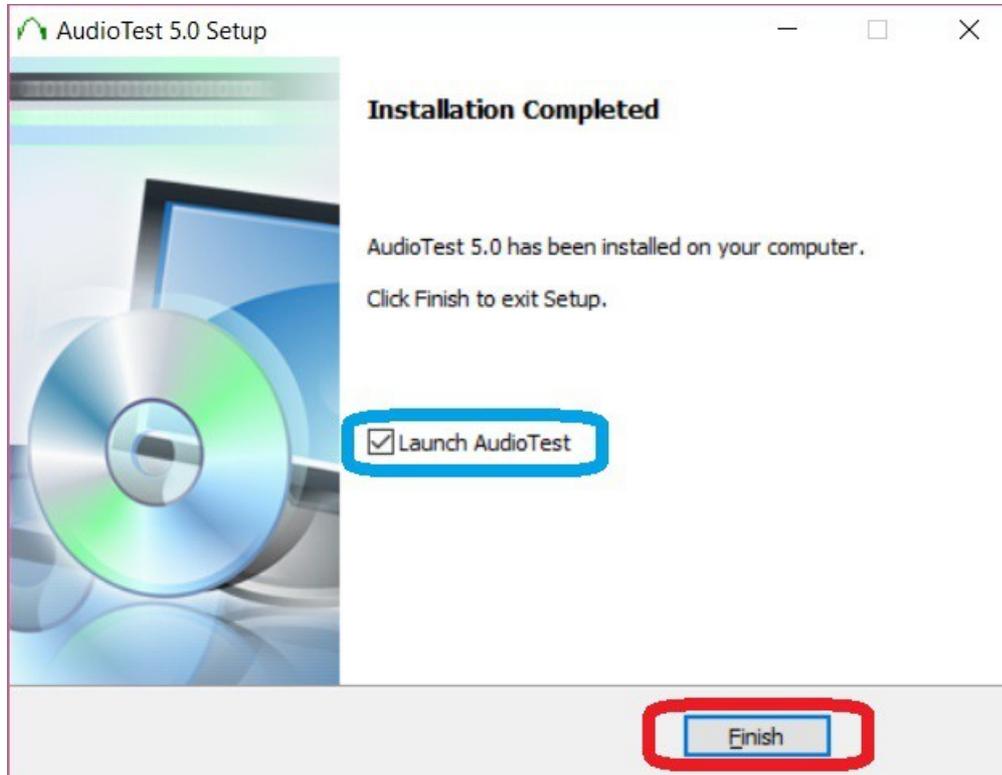

You may launch the program without rebooting the PC.

In the program it is necessary to open the setup once.

This is explained in the audiometry manual.

Feel free to contact us by email, if something is not clear or you have any other question !

Mail to: info@ganseman.com